



The new **i-Go** from Pride Mobility, features advanced folding technology, enabling it to be quickly transported with ease. The i-Go features front suspension, durable seating system, under seat storage and much more. Designed to fit inside any small space, the i-Go is the perfect choice for the active individual.



HI-Torque front suspension













Specifications

Weight Capacity	120 kg (264.5 lbs.)
Maximum Speed ²	Up to 5.95 km/h (3.7 mph)
Ground Clearance	4.4 cm (1.73")
Overall Length ¹	87.63 cm (34.5")
Overall Width ¹	60 cm (23.6")
Turning Radius ^¹	90 cm (35.4")
Standard Controls	Micon Control System
Total Weight (Without batteries)	19.8 kgs (43.6 lbs.)
Battery Weight (each) ³	1.35 kgs (2.9 lbs.)
Suspension	Front
Battery Requirements	2 x 6.6 Ah, Li-ION Battery
Seating	Saddle back seat
Tyres	Front castors: 17.08 cm solid Rear castors: 21.06 cm solid
Colour	Silver

Notes:

- 1. Due to manufacturing tolerances and continual product improvement, this specification can be subject to a variance of + or - 3 %.
- 2. Specification varies with user weight, terrain, battery type, battery condition and tyre pressure.
- 3. Battery weight may vary based on manufacturer.

Note: All specifications subject to change without notice.

The information contained herein is correct at the time of publication; we reserve the right to alter specifications without prior notice. Speed varies with user weight, terrain type, battery charge, battery condition and tyre pressure.

Features

- Large secure storage compartment
- Unique comfort seating
- HI-Torque front suspension
- Micon Control System
- Under seat storage

How To Fold







2 Place joystick in backrest pocket and remove cushion



Fold backrest down, 4 then pullboth arm rests anticlock-wise toward the back



Pull the two black toggles (located on the back) together



Push down on the seat frame until unit fully flattens



6 The i-Go in now ready to go...









