



aspire WALKERS

aspire for...

mobility
comfort
style



ASPIRE WALKER RANGE

Seat Walker Classic 8"

Seat Walker Classic 6"

Seat Walker Mini

Seat Walker Deluxe

Seat Walker XL

Seat Walker Push Down

Tri Wheel Walker

Meal Tray Walker

Meal Trolley




aspire SEAT WALKER CLASSIC 8

✓ DVA Contracted

✓ AS / NZS Approved

✓ Classic 8 is registered in the ARTG, AUST R ARTG 224644

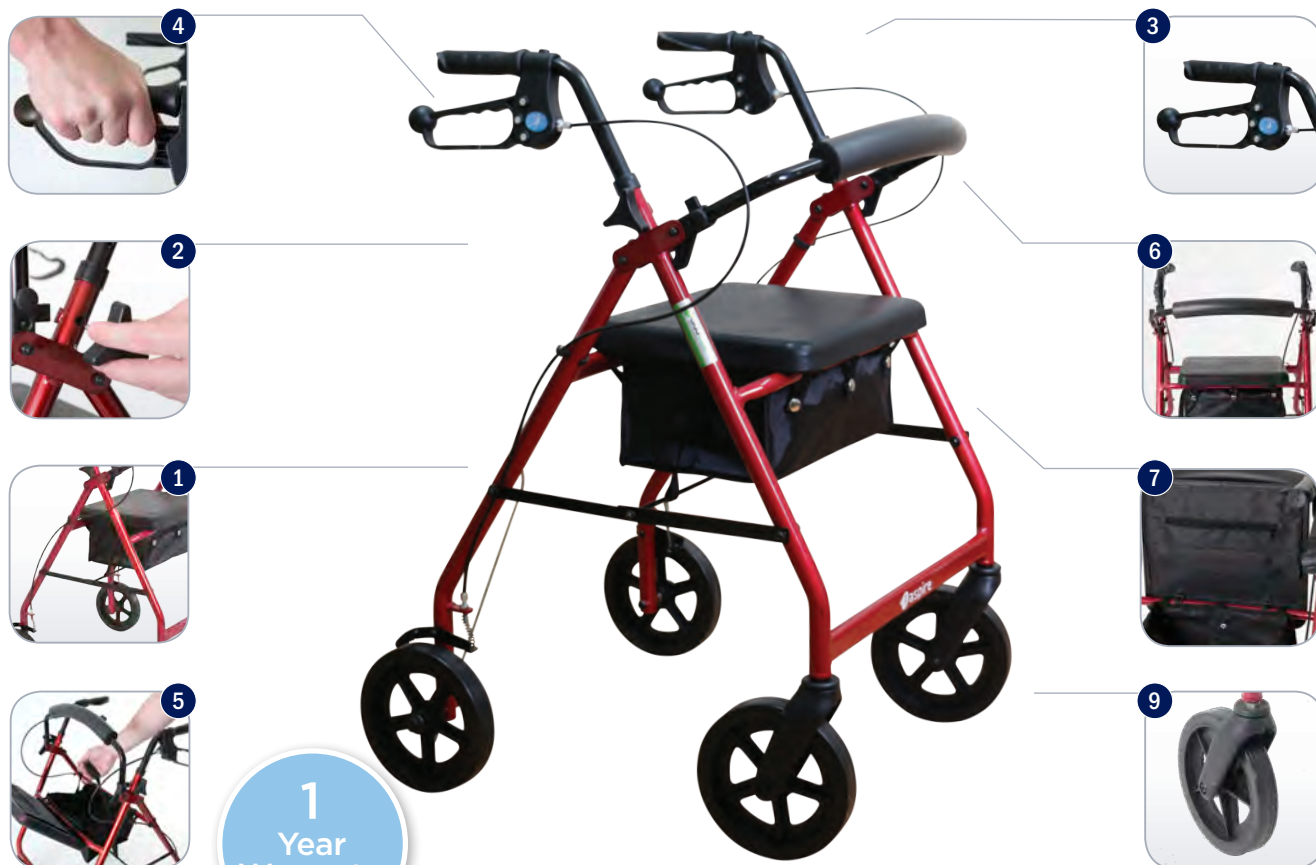

1
Year
Warranty












Available colours:

 WAF705700BL

 WAF705700RE

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Handle height adjusts quickly with hand wheels for optimal sizing
3	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
4	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
5	Padded seat flips up easily for access to storage bag (supplied) or for folding

6	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage
7	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
8	Easy folding mechanism facilitates storage and transportation
9	360° swiveling front wheels increase maneuverability
10	8" solid, puncture-proof wheels which provide increased manoeuvrability over various surfaces
11	Available in blue or red
12	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
										
817 - 972mm	660mm	628mm	305mm	355mm	584mm	815mm	200mm	8"	7kg	130kg

aspire SEAT WALKER CLASSIC 6

✓ DVA Contracted

✓ AS / NZS Approved

✓ Classic 6 is registered in the ARTG, AUST R ARTG 224644



1
Year
Warranty

Available colours:














WAF709600BL



WAF709600RE

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Handle height adjusts quickly with hand wheels for optimal sizing
3	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
4	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
5	Padded seat flips up easily for access to storage bag (supplied) or for folding

6	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage
7	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
8	Easy folding mechanism facilitates storage and transportation
9	Swivelling front castor wheels increase manoeuvrability
10	6" solid, puncture-proof wheels provide increased manoeuvrability over various surfaces
11	Available in blue or red
12	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
										
760 - 915mm	624mm	615mm	305mm	355mm	528mm	769mm	200mm	6"	6kg	130kg

SEAT WALKER MINI

✓ DVA Contracted

✓ AS / NZS Approved

✓ Mini is registered in the ARTG, AUST R ARTG 224651



1
Year
Warranty












Available colours:

 WAF705800BL

 WAF705800RE

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Low 490mm seat height to suit lower height/shorter users
3	Handle height adjusts quickly with hand wheels for optimal sizing
4	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
5	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
6	Padded seat flips up easily for access to storage bag (supplied) or for folding

7	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage
8	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
9	Easy folding mechanism facilitates storage and transportation
10	360° swiveling front wheels increase maneuverability
11	6" solid, puncture-proof wheels provide increased manoeuvrability over various surfaces
12	Available in blue or red
13	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
										
743 - 905mm	556mm	615mm	305mm	355mm	490mm	690mm	200mm	6"	6kg	130kg

aspire SEAT WALKER DELUXE

✓ DVA Contracted

✓ AS / NZS Approved

✓ Deluxe is registered in the ARTG, AUST R ARTG 224650



1
Year
Warranty

Available colours:



WAF705750BL



WAF705750RE

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Inline rear wheels for increased stepping zone
3	Handle height adjusts quickly with hand wheels for optimal sizing
4	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
5	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
6	Padded seat flips up easily for access to storage bag (supplied) or for folding

7	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage
8	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
9	Easy folding mechanism facilitates storage and transportation
10	360° swiveling front wheels increase maneuverability
11	8" solid, puncture-proof wheels provide increased manoeuvrability over various surfaces
12	Available in blue or red
13	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
834 - 989mm	764mm	545mm	305mm	355mm	587mm	836mm	245mm	8"	7kg	130kg

aspire SEAT WALKER XL

✓ DVA Contracted

✓ AS / NZS Approved

✓ XL is registered in the ARTG, AUST R ARTG 224645














1
Year
Warranty

Available colours:

 WAF750020SL

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Wide padded seat and increased 180kg safe working load for large users
3	Handle height adjusts quickly with hand wheels for optimal sizing
4	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
5	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
6	Extra large padded seat flips up easily for access to storage bag (supplied) or for folding

7	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage
8	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
9	Easy folding mechanism facilitates storage and transportation
10	360° swiveling front wheels increase maneuverability
11	8" solid, puncture-proof wheels provide increased manoeuvrability over various surfaces
12	Available in silver
13	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
										
850 - 982mm	697mm	732mm	340mm	495mm	560mm	850mm	230mm	8"	8kg	180kg

aspire SEAT WALKER PUSH DOWN

✓ DVA Contracted

✓ AS / NZS Approved

✓ Push Down is registered in the ARTG, AUST R ARTG 224652



1
Year
Warranty

Available colours: ■ WAF705600BL ■ WAF705600RE

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport	7	Discreet pocket compartment to allow the user to store smaller objects safely e.g. keys
2	Handle height adjusts quickly with hand wheels for optimal sizing	8	Easy folding mechanism facilitates storage and transportation
3	Weight activated brakes ideal for arthritic hands	9	360° swiveling front wheels increase maneuverability
4	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis	10	7" solid, puncture-proof wheels provide increased manoeuvrability over various surfaces
5	Padded seat flips up easily for access to storage bag (supplied) or for folding	11	Available in blue or red
6	Curved and padded backrest offers comfortable seated support and flips up/removes easily for folding/storage	12	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	SEAT DEPTH	SEAT WIDTH	SEAT HEIGHT	BACKREST HEIGHT	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
851 – 962mm	656mm	616mm	305mm	355mm	580mm	815mm	255mm	7"	7kg	130kg

aspire TRI WHEEL WALKER

✓ DVA Contracted

✓ AS / NZS Approved

✓ Tri Wheel is registered in the ARTG, AUST R ARTG 281703










1
Year
Warranty

Available colours:  WAF705500

1	Rustproof, robust aluminium frame provides durability and stability whilst remaining very light to manoeuvre and transport
2	Handle height adjusts quickly with hand wheels for optimal sizing
3	Anatomically designed, latex-free handles allow comfortable grip especially for users with arthritis
4	Ultra-soft hand brakes with palm ball lock effortlessly, offering increased comfort for users with arthritis or wrist/hand joint pain
5	Storage bag included as standard

6	Meal Tray & Basket included as standard
7	Easy folding mechanism facilitates storage and transportation
8	360° swiveling front wheel increases maneuverability
9	8" solid, puncture-proof wheel provide increased manoeuvrability over various surfaces
10	Available in red
11	Easy wipe-clean surfaces

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	FOLDED WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
 800 - 920mm	 730mm	 645mm	 275mm	 8"	 5.5kg	 115kg

aspire MEAL TRAY WALKER

✓ DVA Contracted

✓ AS / NZS Approved

✓ Meal Walker is registered in the ARTG, AUST R ARTG 271256



1
Year
Warranty

Available colours:  DLG272061

1	Sturdy & robust walking aid for use within the home, providing a means of transporting meals or other items from one room to another	4	Clip on plastic trays with outer lip edge to capture spillage and prevent items from slipping off
2	Large 6" wheels make it easier to negotiate carpets and door thresholds	5	Height adjustable handles
3	Lockable hand brakes provide additional safety when rising from a seated position	6	Easy clean frame & food trays
		7	Tool free assembly

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	LOWER TRAY HEIGHT	UPPER TRAY HEIGHT	TRAY WIDTH	WHEEL SIZE	UNIT WEIGHT	S.W.L
								
770 - 920mm	560mm	585mm	430mm	696mm	450mm	6"	8kg	125kg

aspire MEAL TROLLEY

✓ DVA Contracted

✓ AS / NZS Approved









✓ Meal Trolley is registered in the ARTG, AUST R ARTG 271256




1
Year
Warranty

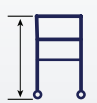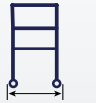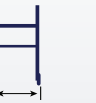Available colours:  DLG272060

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1	Sturdy & robust meal transport trolley, providing a means of transporting meals or other items from one room to another	4	Height adjustable in one inch increments
2	3" castors with two locking to hold the trolley in place	5	Easy clean frame & food trays
3	Clip on plastic trays with outer lip edge to capture spillage and prevent items from slipping off	6	Tool free assembly

OVERALL HEIGHT	OVERALL LENGTH	OVERALL WIDTH	LOWER TRAY HEIGHT	UPPER TRAY HEIGHT	TRAY WIDTH	CASTOR SIZE	UNIT WEIGHT
							
810 - 937mm	503mm	460mm	351 - 478mm	615 - 740mm	450mm	3"	6kg

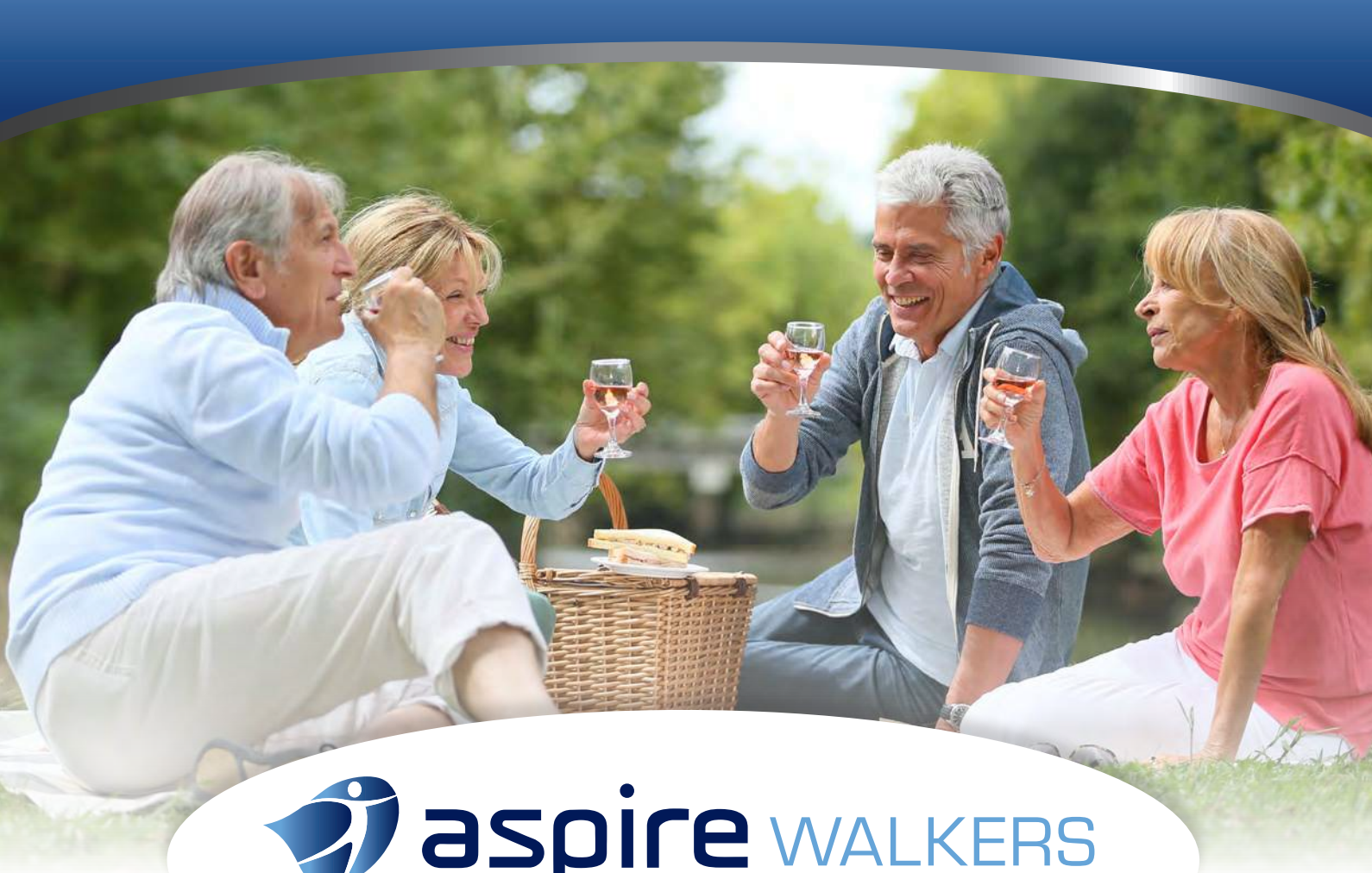

aspire WALKERS *vogue*



Lightweight White

Carbon Super Lite

Lightweight Red

Lightweight Forearm

The stylish European designed Aspire Vogue Walkers are a lightweight, versatile range with a sturdy frame and easy to fold locking system, suitable for compact transportation when folded. Manufactured with the utmost attention to quality, including soft ride wheels, advanced deluxe brakes, ergonomic handles and a forearm option.

Enjoy your independence with peace of mind, in style.


ASSOCIATED PRODUCTS



LIFTERS & SLINGS




TRANSFER AIDS

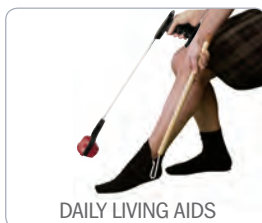

DAILY LIVING AIDS




THERAPY & EXERCISE

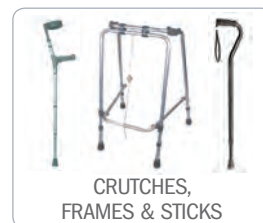

CRUTCHES,
FRAMES & STICKS



SCOOTERS

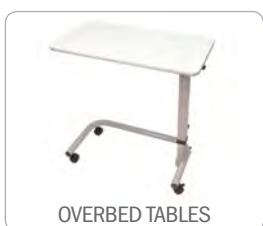

OVERBED TABLES



BED STICKS & POLES



FOAM MATTRESSES



AIR MATTRESSES



DAY CHAIRS



LIFT & RECLINE CHAIRS

Distributed by:

**power
mobility**
The power to move you

07 3265 4663

2/452 Bilsen Rd, Geebung QLD 4034
sales@powermobility.com.au • powermobility.com.au

ACCESSORIES




OXYGEN BOTTLE
HOLDER
WAA686905



WALKING STICK
HOLDER
WAA686925



CLIP ON WALKER
BAG
STD: WAA686915

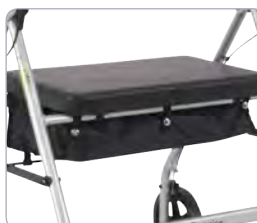

CLIP ON WALKER
BAG
XL: WAA686920



SEAT WALKER
BASKET
WAA686910

ASBR02.24 100118

