

1

Dissassemble your travel commode before packing. Place one wheel in the base of your bag.

4

Place second wheel over folded backrest.

2

Push down on the levers to fold the rear legs and place your frame in the bag. The frame should face forward as pictured.

5

Place seat on top of wheel as pictured.

3

Place folded footrest over frame as pictured, then fold backrest down.

6

Place castor arms along the sides of the bag. Store any remaining accessories in the available space.
Place axles and pins in pocket in the lid of the bag



TRAVEL COMMODE

ASSEMBLY MANUAL

CONTACT


1 Chinook Place, Hornby South
Christchurch, New Zealand

tukidesign.co.nz
info@tukidesign.co.nz
+64 3 323 6478

ASSEMBLING YOUR COMMUNE


1

Fully unfold the backrest then unfold the rear legs one at a time until the backrest is locked in place.

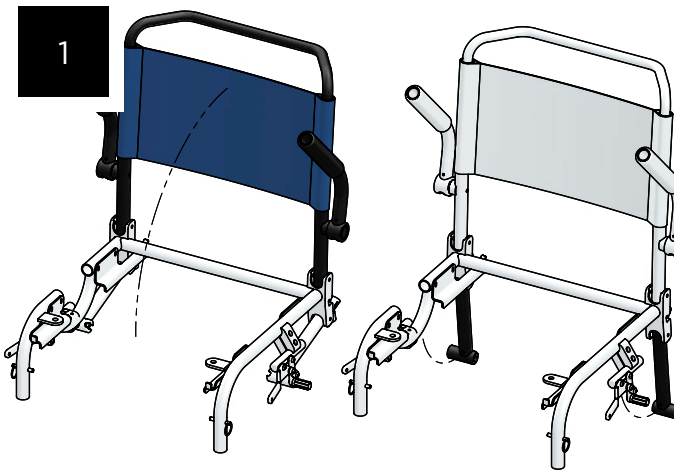


2

Slide the footrest crossmember over the front legs. Insert castor arms facing forward and out, lining up the stickers on the frame. Secure with the provided locking pins.

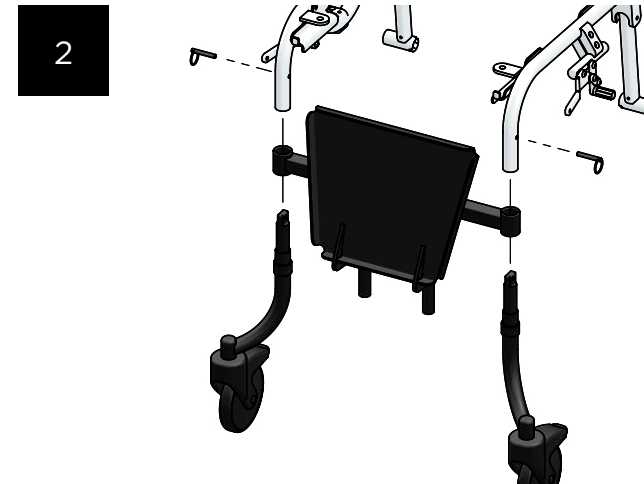


3a

If using self-propelling rear wheels, place axles (from zippered pocket in bag) through wheels into axle receivers, and secure with locking pins.

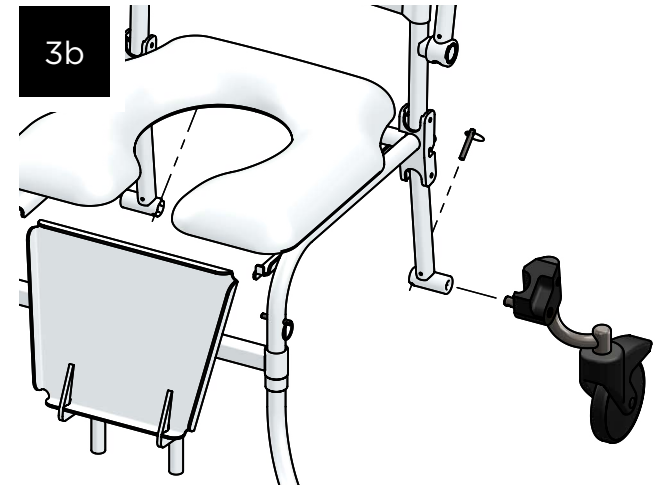

4

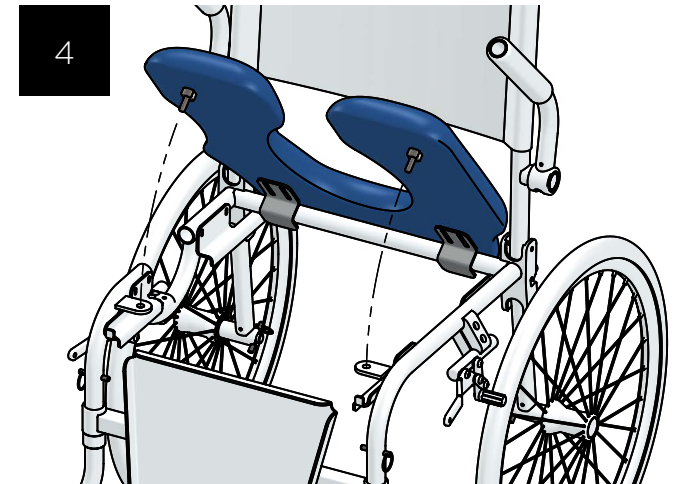


3b

If using rear castors, slide castor blocks into axle receivers and secure with locking pins. Make sure that castors are oriented correctly as shown.


4

Slide seat over frame cross bar. Ensure front pins locate correctly into pin receivers.

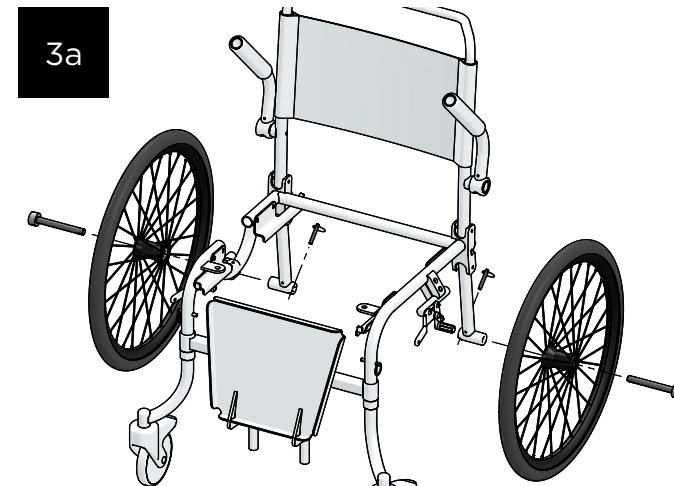

3a

5

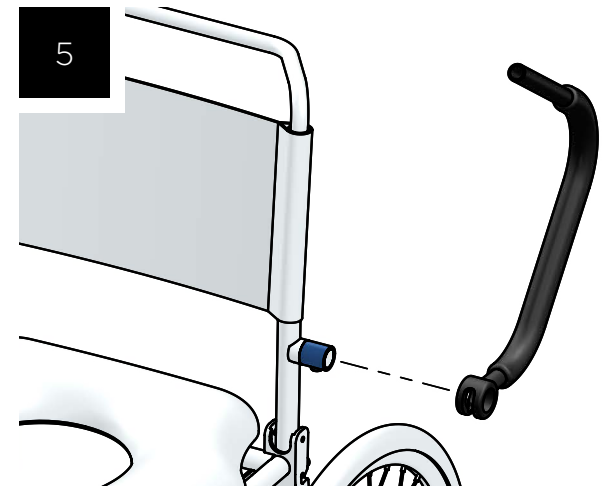

5

If using removable locking armrests, slide the armrests onto the pivots as shown.